

Get Started with Q-interactive

Q-interactive Start-Up Guide




Here are instructions for installing the Q-interactive® Assess app, establishing the proper settings on both iPad® devices, finalizing your login credentials, and creating a practice assessment session. We recommend practicing the tests a few times before assessing real clients. Please review the [Q-interactive Training Modules](#), as it provides an in-depth walkthrough of both set-up and use of Q-interactive.

SECTION 1: Download the App



The Q-interactive Assess app must be downloaded onto both iPads in order to use the system.

STEP 1

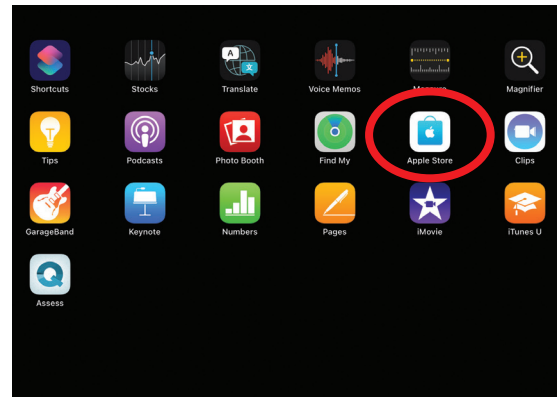

Open the App Store, found on the home screen of the iPad.

STEP 2

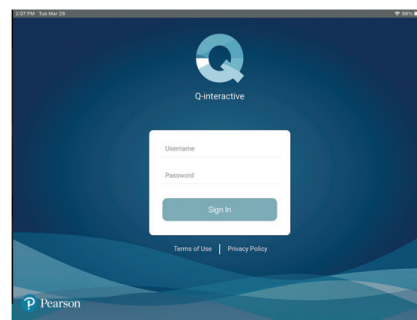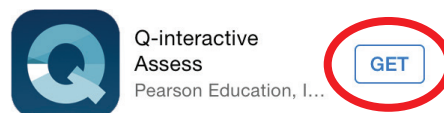

Search for Q-interactive and tap on "GET."


SECTION 2: Configure the iPads

With Assess downloaded, let's configure the settings on both devices.

This is an essential step for a few reasons:

- Ensure iPads pair correctly in order to capture data from each other.
- Stimuli display in a standard manner.
- Assist in the smooth operation of Q-interactive.
- Eliminate distractions, which may occur during the assessment.

STEP 1

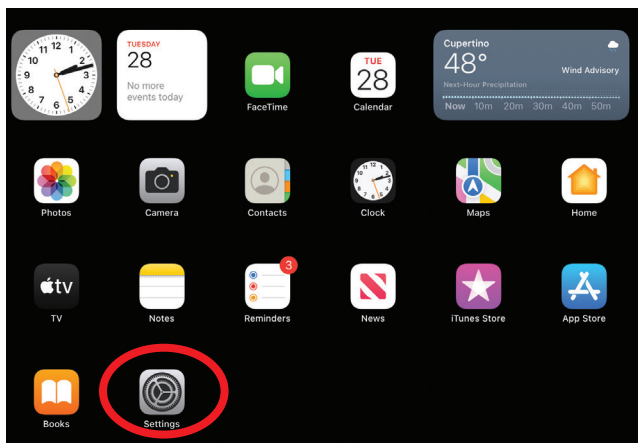

Touch the Settings app located on the home screen.


STEP 2. Adjust your iPads according to the following table. Q-interactive Assess uses Wi-Fi to connect the Practitioner and Client devices. You must be connected to an active Wi-Fi network to download and sync assessments. You do not need to be connected to an active Wi-Fi network to administer assessments, but you do need to have Wi-Fi enabled on both devices.

Practitioner's iPad	Client's iPad	Setting	Menu	Choice
•	•	Wi-Fi	Enable Wi-Fi	Select Network
	•	Display & Brightness	Brightness	Slide bar to far right
	•	Display & Brightness	Night Shift	Off
	•	Display & Brightness	True Tone	Disable True Tone
•	•	Display & Brightness	Auto-Lock	15 Minutes
	•	Accessibility	Display & Text Size	Disable Auto-Brightness

SECTION 3: Creating an Assessment Session

Your Welcome email gets this started by first establishing your login credentials.


STEP 1

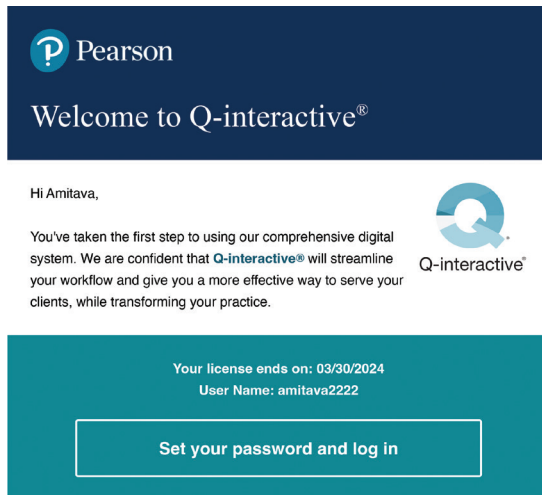

The “Click here” link will open the screen to establish your Q-interactive password.


After submitting the password and accepting the Terms and Conditions of Q-interactive, you are now ready to create a practice assessment session.

STEP 2

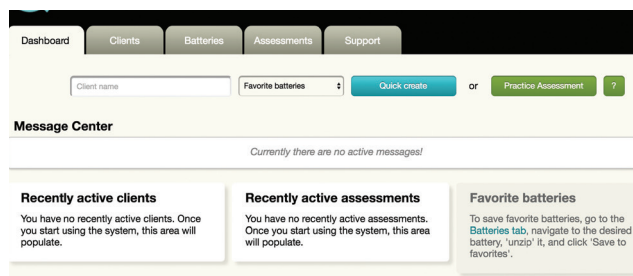


From the Dashboard tab, click the Practice Assessment button to make the Practice Client profile and Save.

STEP 3

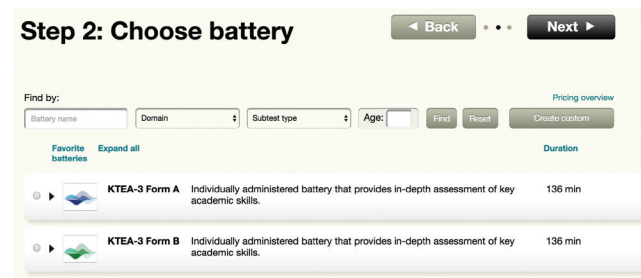

Select battery by clicking the button to the left, then click the Next button.

STEP 4

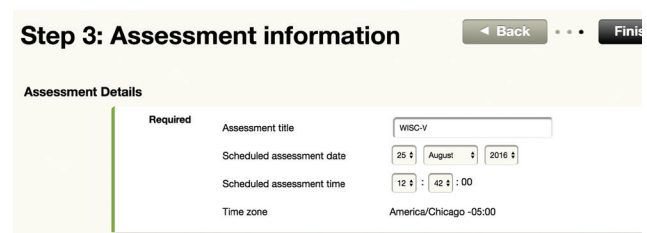

Set the date and time for administration and click the Finish button


STEP 5




Click the Send to iPad button and you'll be all set to log into the iPads!

Login to both iPads with the same login credentials established earlier to begin testing.

STEP 6

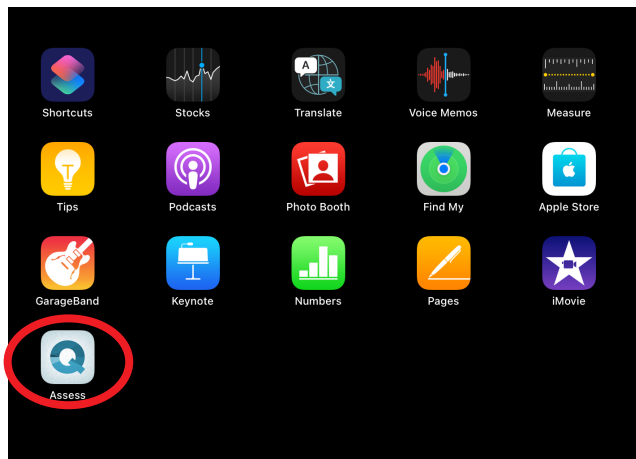


STEP 7

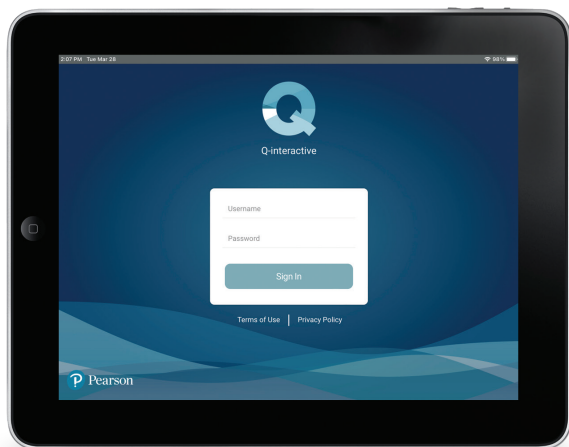

Enter your User Name and Password. Allow 10-20 minutes for your complete library of tests to download.

Your iPads will connect when you start your assessment and both iPads have been configured correctly.

STEP 8

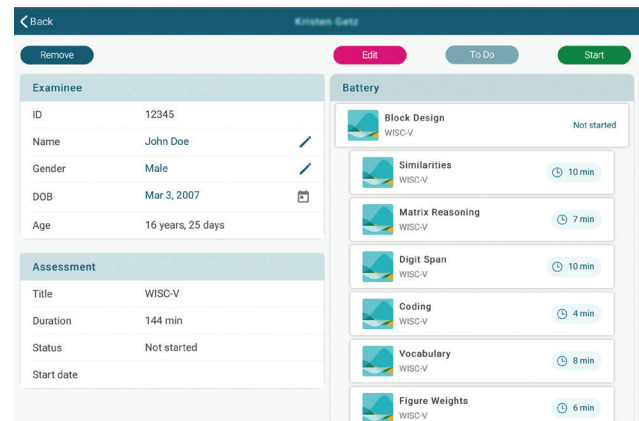

Tap the Start button to begin the assessment.
Practice assessment sessions have all the functionality of real assessment sessions, except all scores default to zero.

Congratulations! You are now ready to use Q-interactive!

Please reference the [Q-interactive Training Modules](#) for detailed instructions.

800-627-7271 | PearsonAssessments.com

Copyright © 2024 Pearson Education, Inc. or its affiliates. All rights reserved. Pearson and Q-interactive are trademarks, in the US and/or other countries, of Pearson plc. iPad is a registered trademark of Apple Inc. C11080 EL 10/24